

AM3 Chair Installation

Simple 8 Steps to Assemble and
Install your AM3 Dental Chair

Step One

- Remove Chair from Pallet



Step Two

- Gently Fold Seat Back To Meet Chair Top Motor Shaft



Step Three

- Insert Seat Back Bolt Thru Yoke and Motor Shaft




Step Four

- Install Seat Back Nut on Seat Back Bolt
– Tighten with Wrench and Allen Wrench
- **DO NOT OVER TIGHTEN**



Step Five

- Completed Motor Shaft Assembly

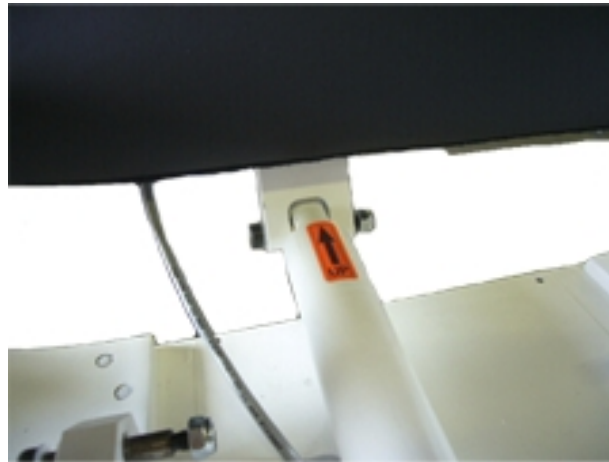

Step Six

- Remove Chair Top Seat Cushion From Shipping Box



Step Seven

- Install Seat Cushion Bolts and Thread on Locking Nuts. **DO NOT OVER TIGHTEN**



Step Eight

- Install Seat Cushion Side Panels with Snaps and Velcro. AM3 Chair is Assembled

