

California Chair Installation

Simple 10 Steps to Assemble and
Install your California Dental Chair

Step One

- Remove Chair Base from Pallet



Step Two

- Remove Chair Top from Box



Step Three

- Remove Nuts and Lock Washers from Chair Top Bolts



Step Four

- Set Chair Top on Chair Base
- Align Chair Bolts into Holes




Step Five

- Plug Chair Top Connector into Chair Base Connector – Make sure they lock.
- Insert into Chair Frame



Step Six

- Plug Chair into Power Source and Raise Chair

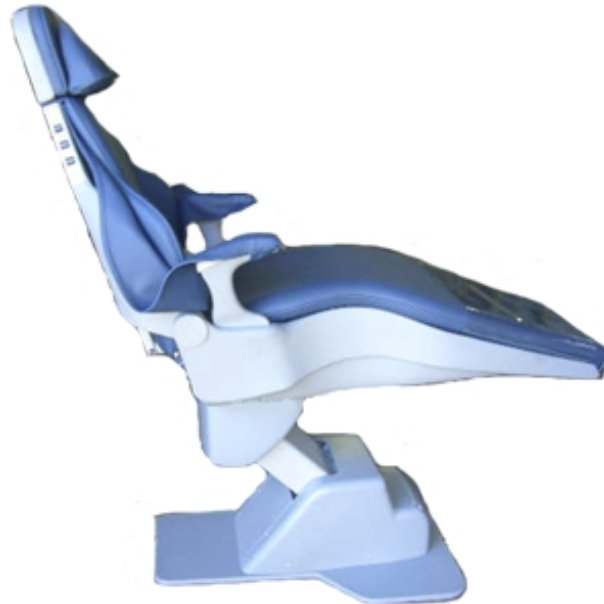


Step Seven

- Install Chair Nuts on to Chair Bolts Protruding thru Metal Plate.
- Tighten with Wrench



Step Eight

- Remove Upper Base Cover with Phillips Screwdriver – 4 Screws

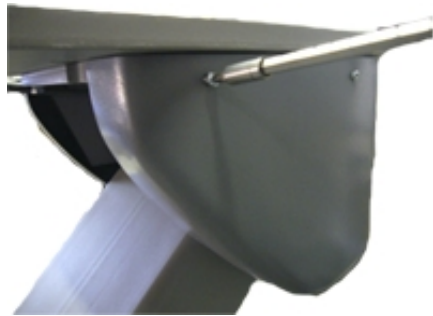


Step Nine

- Install Chair Top Nuts on Rear Chair Top Bolts
- Tighten with Wrench



Step Ten

- Re-Install the Upper Base Cover with Phillips Screwdriver – 4 Screws
- DO NOT OVER TIGHTEN

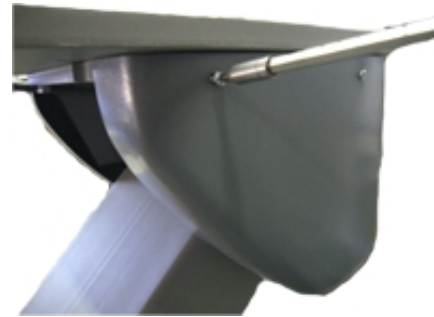

- California Chair is Completely Assembled