

# Diamond Dental Light Installation

Simple 10 Steps to Assemble and Install your Diamond Dental Light

# Step One

- Remove Light Arm from Box. Careful the Light Arm is Spring Loaded.



# Step Two

- Thread the Light Cord Thru the Post. **Make sure your light bushing is in place.**




# Step Three

- Slide Light Arm Into Light Bushing in 2" Post. It will be tight. You choose to lubricate to ease it in.



# Step Four

- Light Arm Completely in the Post.

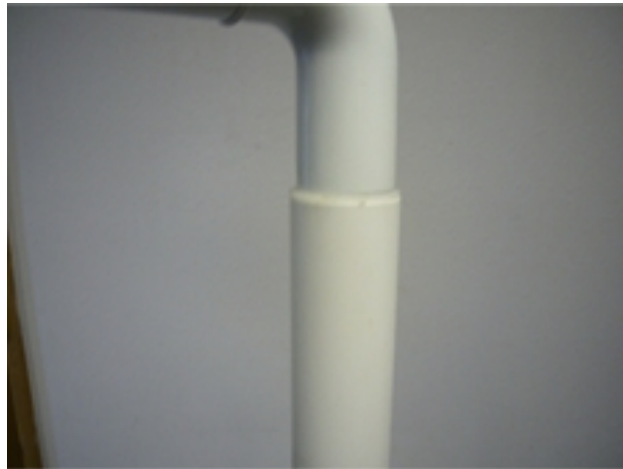

# Step Five

- Remove Light Head from Box

*Light  
Collar*



# Step Six

- Plug Connectors Together Firmly.

*Special Note:  
Place Light  
Collar on  
Prior to Installing  
Light Head*



# Step Seven

- Slide Light Head into Light Arm Housing.



# Step Eight


- Line Up Light Head Screw Hole with Light Arm Housing Screw Hole and Thread Retaining Screw Completely.



*Slide Light  
Collar over  
Retaining  
Screw*

# Step 9

- Install Transformer Under Mounting Plate with Screw. Plug Light Cord and Transformer Plugs together. Make Sure Pins Line Up.

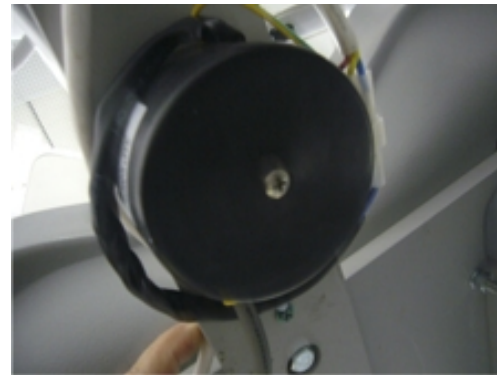

# Step 10 – Final Step

- Plug Electrical Cord into Chair Outlet.

